

Laboratorium Medische Immunologie (Amsterdam UMC, locatie AMC)

Assay performance data:

**Tetanus toxoid As titer bepaling in serum of plasma**

Traceerbaarheid:

- Info uit bijsluiters: The assay is calibrated against the NIBSC tetanus antitoxin reference preparation, 76/589 which is supplied by the National Institute for Biological Standards and Control (NIBSC: [www.nibsc.ac.uk](http://www.nibsc.ac.uk)).

Referentie-interval: Beschikbaar via contactpersonen van deze bepaling

- Bron ref-interval: door laboratorium zelf vastgesteld in jong volwassen gezonde controles
  - Geslachtsafhankelijkheid: Nee
  - Leeftijdsafhankelijk: Ja want ze zijn afhankelijk van vaccinatie geschiedenis

Stabiliteit monster bij

KT: minimaal 3 dagen  
4°C: minimaal 7 dagen  
-20°C: minimaal 1 jaar

Imprecisie

- CV tot ca. 15%

Meetbereik:

- Bij een uitslag boven het meetbereik wordt de test herhaald op een extra verdund monster waardoor er uiteindelijk altijd een kwantitatieve uitslag volgt.

Deelname aan extern QC-programma? Ja, UK-NEQAS Specific Microbial Antibodies